



Celara[®]

DENTURE SYSTEM

Dentures predictably delivered in 2-3 visits with minimal chairtime.

- Quickly and easily provides a 3D blueprint of the patient's existing denture and oral cavity.
- Gives the clinician a starting point and allows modification of any aspect of the existing denture.
- Minimal adjustments and increased patient satisfaction.
- Increased profitability and productivity for your practice

Step 1



Making an Impression

A final impression is made in the existing denture: border molding with heavy body impression material, and then a light wash final impression.

Step 2



Boxing Impressions

Quickly and easily box the denture using the Celara Disposable Container, Celara Extended Pour Alginate, and Celra Quick Set Stone. The patient is only without their denture for 20 to 25 minutes in your office, and leaves with their denture after the first visit.

Step 3



Wax Pattern Fabrication

The void created when the denture is removed is then injected with Celara Wax in the dental laboratory or dental office using the Celara Wax Injector. This results in a perfect wax pattern that is used to provide an accurate set-up for a predictable try-in on the second patient visit.

Celara Technical Support: 800-823-5272

Celara is a Registered Trademark of Dentovations. US Patent #6,224,375 and other patents pending
www.celara.net

Another quality product by **dentovations[®]**